## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER

The intended user should be the patient, a licensed healthcare professional, or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

### INTENDED USE/INDICATIONS

- DonJoy Actify Light is designed to provide support and stabilization to the ankle.
- Mild to moderate sprains of the lateral ligament.
- Entorses légères à modérées du ligament latéral.
- Blessures syndesmotiques légères (grade 1).
- Initial management of chronic ankle instabilities.
- Return to sports, prevention of re-injury.
- Ligament instability.
- Sprain prevention.

### CONTRA-INDICATIONS

- Do not use if you are allergic to any of the materials contained in this product.
- Peripheral arterial diseases.
- Impaired blood flow in the leg (e.g. with diabetes mellitus).
- Impaired lymphatic drainage or soft tissue swelling of unknown origin.

### PERFORMANCE CHARACTERISTICS

- Providing support and stabilization to the limb or body segment.
- Providing partial immobilization or controlled movement to the limb or body segment.
- Providing mild protection of a limb or body segment for acute and prophylactic care.

### WARNING AND PRECAUTIONS

We recommend that the first application be made under supervision of a healthcare professional. If you have skin conditions/trauma in the relevant part of the body, the product should be fitted and worn only after consultation with a healthcare professional.

If you experience any discomfort, pain, swelling or sensation changes, or any unusual reactions while using this product, consult your healthcare professional immediately.

This product is for single patient use only.

Do not use this device if it was damaged and/or packaging has been opened.

The product should be removed during long periods of rest.

It is not recommended to drive while wearing the brace as your ability to brake may be impaired.

Do not use showering or bathing.

Keep the product away from children.

Do not use on open wound(s) or/der sensitive skin.

Do not use this product if it is damaged or defective.

Do not use this product in any way that may damage its integrity or use the device other than intended.

The frequency and duration of use should be determined by a healthcare professional.

Do not fit the brace too tightly, as this could lead to local pressure symptoms.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising to use of this device.

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### APPLICATION INSTRUCTIONS

The product is designed to be worn during everyday activities and sports. It can be worn over a sock. We recommend wearing a closed shoe with the brace for additional support and optimal stabilization. The brace can be used with or without bilateral removable stays.

- Unfasten the twin straps.
- Open the lace locking mechanism, loosen the laces and open the brace.
- In the event of discomfort in the ankle bone area, the position of the bilateral removable stays can be modified by sliding them above or below the switch located at the bottom of the brace. Recommended stay position: lateral (external) stay at the lowest position and medial (internal) stay at the upper position.
- The bilateral stays can be removed completely, by popping their tips out of the guided pockets.
- Unfasten the heel strap, and slide your foot into the back of the brace.
- Pull the laces to close the brace without tightening excessively to avoid any tourniquet effect.
- Close the lace locking mechanism in place.
- Cross the twin straps over the top of the foot once, then cross them a second time at the base of the calf before attaching the self-fastening tabs to the sides of the ankle brace.

### CLEANING INSTRUCTIONS

Machine washable (40°C) at gentle cycle. Hook and loop of the straps should be closed for washing to avoid any damage. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not iron.

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